



BATH BENCHES

MODEL NO.	102	Bath Bench w/Back
MODEL NO.	202	Bath Bench (no back)
MODEL NO.	401	Bariatric Bath Chair w/Back
MODEL NO.	402	Bariatric Bath Chair (no back)

Bath Bench w/Back



Bariatric Bath Chair w/Back



Bath Bench (no back)



Bariatric Bath Chair (no back)



NOTE: Check ALL parts for shipping damage before use. In case of shipping damage, DO NOT use. Contact Carrier/Dealer for further instruction.

Index

1. General Warnings.....	3
2. Installation Warnings.....	3
3. Leg Height Adjustment.....	3
4. Installing Back.....	4
5. Limited Warranty.....	5

GENERAL WARNINGS

DO NOT install this equipment without first reading and understanding this instruction sheet. **IF** you are unable to understand these Warnings, Cautions and Instructions, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment otherwise, injury or damage may occur.

Rubber Tips **MUST** be in contact with tub floor at all times.

Always inspect Bath Bench and Bariatric Bath Chair to ensure that it is properly positioned and stable before using. **DO NOT** use if Bath Bench or Bariatric Bath Chair is wobbly or unstable.

Check rubber tips for rips, tears, cracks or wear. If any of these conditions exist, replace rubber tips immediately.

Users with limited physical capabilities should be supervised or assisted when using the Bath Bench or Bariatric Bath Chair.

The Bath Bench has a weight capacity of 330 lbs.
The Bariatric Bath Chair has a weight capacity of 550 lbs.


The Bath Bench and Bariatric Bath Chair is **NOT** to be used as a transfer bench or transfer device.

INSTALLATION WARNINGS

Ensure that the snap buttons fully protrude through the same respective adjustment hole of each adjustable leg before using. This ensures that the adjustable legs are securely locked into position and an even height adjustment is achieved.

After **ANY** adjustments, repair or service and **BEFORE** use, make sure that all attaching hardware is tightened securely.

LEG HEIGHT ADJUSTMENT

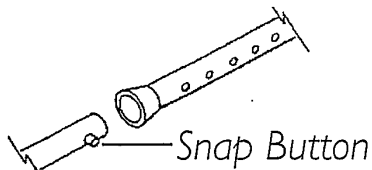

1. Select desired height for Bath Bench and Bariatric Bath Chair by aligning snap button of each leg extension with appropriate adjustment hole in leg frame.

2. Ensure snap buttons are fully engaged into adjustment hole of each leg frame and are at the same height before using.

INSTALLING BACK

1. Take Back of Chair/Bench and depress snap buttons. Slide through first loop.
2. Align back support poles with the second loop. Depress snap buttons and slide into second loop.
3. Make sure snap buttons are fully engaged to ensure back is securely in place.

LIMITED WARRANTY

PMI WARRANTS THIS PRODUCT TO BE FREE FROM DEFECTS IN MATERIALS AND WORKMANSHIP.

THIS WARRANTY IS EXTENDED ONLY TO ORIGINAL PURCHASER/USER OF OUR PRODUCTS.

PMI WARRANTS ITS PRODUCTS TO BE FREE FROM DEFECTS IN MATERIAL AND WORKMANSHIP FOR A PERIOD OF 2 YEARS OF USE BY ORIGINAL PURCHASER. IF WITHIN SUCH WARRANTY PERIOD ANY SUCH PRODUCT SHALL BE PROVEN TO BE DEFECTIVE, SUCH PRODUCT SHALL BE REPAIRED OR REPLACED, AT PMI'S OPTION. THIS WARRANTY DOES NOT INCLUDE ANY LABOR OR SHIPPING CHARGES INCURRED IN REPLACEMENT PART INSTALLATION OR REPAIR OF ANY SUCH PRODUCT. PMI'S SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO SUCH REPAIR AND/OR REPLACEMENT. FOR WARRANTY SERVICE, PLEASE CONTACT THE DEALER FROM WHOM YOU PURCHASED YOUR PMI PRODUCT. IN THE EVENT YOU DO NOT RECEIVE SATISFACTORY WARRANTY SERVICE, PLEASE WRITE DIRECTLY TO PMI AT THE ADDRESS ON THE BACK PAGE. PROVIDE DEALER'S NAME, ADDRESS, MODEL NUMBER, DATE OF PURCHASE, INDICATE NATURE OF THE DEFECT AND, IF THE PRODUCT IS SERIALIZED, INDICATE THE SERIAL NUMBER.

PMI WILL USE A RETURN AUTHORIZATION. THE DEFECTIVE UNIT OR PARTS MUST BE RETURNED FOR WARRANTY INSPECTION USING THE SERIAL NUMBER, WHEN APPLICABLE, AS IDENTIFICATION WITHIN 30 DAYS OF RETURN AUTHORIZATION DATE. DO NOT RETURN PRODUCTS TO US WITHOUT PRIOR CONSENT. C.O.D. SHIPMENTS WILL BE REFUSED: PLEASE REPAY SHIPPING CHARGES.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAIL TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT PMI'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS, PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT SPECIFIC CONSENT OF PMI; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND PMI'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN

AN AUTHORIZED PMI DEALERS). SUCH EVALUATION SHALL BE SOLELY DETERMINED BY PMI.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. PMI SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

ALL PRODUCTS LEAVE OUR WAREHOUSE IN BRAND NEW CONDITION. IT IS THE CUSTOMER'S RESPONSIBILITY TO EXAMINE ALL SHIPMENTS FOR DAMAGE IMMEDIATELY UPON ARRIVAL. COUNT THE PACKAGES AND SIGN FOR SAFE DELIVERY. SIGN ONLY FOR WHAT YOU RECEIVE. IF MERCHANDISE IS DAMAGED CALL CARRIER AND SECURE DAMAGE INSPECTION REPORT YOU HAVE 15 DAYS TO SECURE DAMAGE REPORTS OTHERWISE YOU MAY LOSE YOUR PRIVILEGE OF FILING A CLAIM FOR LOSS OR DAMAGE.

FREIGHT POLICY: PLEASE CONTACT YOUR SALES REPRESENTATIVE FOR DETAILS.

For Service Call Your Authorized **PROBASICS** Medical Dealer:



15 South Main Street
2nd Floor
Marlboro, New Jersey 07746
www.pmiusa.biz

